# R&S

## Polishing paste

- (fr)** Pâte prophylactique goût mentholé
- (en)** Prophylaxis paste mint flavour
- (es)** Pasta profiláctica sabor menta
- (pt)** Pasta profilática sabor mentolado
- (it)** Pasta profilattica gusto mentolato
- (de)** Polierpaste mit Pfefferminzgeschmack
- (nl)** Profylactische pasta met mintsmaak

**(fr)**

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**(en)**

Contains only natural flavourings and no fluoride. The blend of abrasives used in the various grades are selected to maximise the removal of plaque and tartar without appreciable abrasion of enamel. The thixotropic properties are carefully controlled to ensure easy transfer to rubber cup or brush without undesirable spattering at the rotation speeds used in prophylaxis.

**Indications:** Use for the removal of plaque and tartar from patients teeth.

**Directions for use:** The prophy paste is thixotropic, so, if desired fluidity can be increased by gentle spatulation before use. Replace lid after use. For best results, use a prophy type handpiece and rubber cup/cup brush with paste and run

at low speed over each tooth surface, making sure that the edge of the rubber cup/cup brush goes below the free margin of the gingiva into the gingival sulcus. Repeat this procedure for each tooth until all exposed surfaces have been treated. Unwaxed dental floss may be used to remove any excess paste through interproximal areas.

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Advanced Healthcare Ltd
Advena Ltd.