

91%
overall
rating

CFPM

www.rs-dental-products.com/

Description

R&S TURBOPRINT CLASS A is a rapid-setting, elastic, dust-free alginate. It is appropriate for use in a wide range of impression procedures in routine dental practice. Total time from start of mix to removal from the mouth is 2:40; intraoral time is 1:00. A standardized particle size is designed to provide rapid water absorption and a uniform, smoother, compact surface. Pouring of an impression may be delayed for several hours without compromising its accuracy if it is stored in a sealed bag. The product is mint flavored and has a blue color. **R&S TURBOPRINT CLASS A** is available in a 500-g single bag. Unopened bags have a four-year shelf life. Twenty-nine consultants evaluated **R&S TURBOPRINT CLASS A** making 361 impressions during a six-week period. This product received a 91% clinical rating.

Clinical Tips

- Cleanup from the soft tissue is easier if the skin and tongue are moist rather than very dry.

"STACKS
WELL IN
THE TRAY."

Consultants' Comments

- "Resists tearing."
- "Adheres to the tongue and face."
- "The mint taste is not very strong."
- "I'd like a faster setting time."

Product Features

- Ninety-five percent of the consultants rated **R&S TURBOPRINT CLASS A** "as good as" or "better than" the alginate they were currently using. They found it created little dust, was easy to dispense, mix and transfer to a tray.
- Some dental assistants commented that the alginate seemed grainy while it was being mixed but it didn't affect the final result.
- Patients found the mild flavor and scent to be pleasant.
- It provides a high quality of detail in the impressions and on the poured models.
- Sixty-seven percent of the consultants would purchase **R&S TURBOPRINT CLASS A** and eighty-seven percent would recommend it to colleagues.

