

**96%**  
overall  
rating

CFPM

www.rs-dental-products.com

### Description

**R&S Turbocclusion** is an excellent fast-setting, addition silicone bite registration material. The working time is 30 seconds and the setting time is one minute from the start of mix. The package contains two, 50-ml automix cartridges and 12 mixing tips. Twenty-four consultants used **R&S Turbocclusion** in 341 clinical applications.

This product received a **96% clinical rating**.


### Consultants' Comments

- "Very hard, rigid, and accurate material."
- "Dispenses evenly – no mess."
- "Patients like the fast set."
- "Requires a lot of pressure to express from the cartridge."


"I HAD FEWER  
ADJUSTMENTS WITH  
R&S TURBOCCLUSION  
BITE REGISTRATIONS."



Compared to Competitive Products:

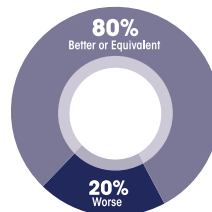

Percentage of Consultants Who Would:

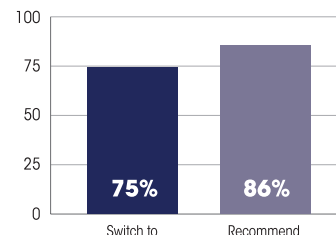

### Unique Features

**R&S Turbocclusion** was rated very good to excellent for ease of dispensing and application, viscosity, ease of removal, and detail of impression. The material flows evenly onto tooth surfaces without slumping. The set bite registration exhibits good detail and can be trimmed with stones or acrylic burs; it is too hard and brittle to be trimmed with a scalpel blade. No rebound occurs when the casts are articulated. Eighty percent of consultants rated **R&S Turbocclusion** the same as or better than the bite registration material they were currently using; 75% of consultants would purchase it, and 86% would recommend it.