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### Instructions for use

EN

Before using the STERILE GATES DRILLS, please see the IFU as below

#### Warnings

- Only skilled dentists are allowed to use.
- Do not use this product except for the dental service and treatment
- Use it in accordance with the intended use.

#### Contraindications and prohibitions

Do not use this product for a patient who indicates sensitization and allergic reaction.

#### Structures and principals

1. This product is used by connecting to active device and rotates to enlarge the root canal orifice. It is used by connecting to an active device.
2. Main materials:  
Working cutting part and shank are made in stainless steel, this includes nickel and chromium.

#### Possible Combinations with active device

1. Use the active device which can hold the shank precisely. (Diameter of Contra Angle shank: 2.35 mm).
2. Use the active device which could control at allowable engine speed.

#### Precautions for use

1. To prevent infection, if you reuse the product sterilize the product by autoclave and make sure sterilization is completed before using.
2. Keep the allowable engine speed strictly as over speed could cause fractures and injury.
3. Follow the instructions of active device manufacturer strictly and insert the shank all the way to avoid its half-way chucking.
4. The active device should be used in accordance with its instructions
5. Choose the most appropriate type for each case and follow the general method.
6. Before using, make sure the instruments outside of oral cavity that there are no deformations, scratches and cracks.
7. If the head of product is thin, long or large, there are possibilities for breaking or twisting. Because of this, be sure to avoid using unreasonable angle and excessive pressure.
8. Wear safety glasses to protect your eyes from damage. Also, wear a dust protective mask to prevent inhaling dust.
9. Wear rubber dam to avoid accidental ingestion and falling.
10. Do not use this instrument for any purposes except for listed applications above.
11. Only for use by dentists.
12. This product should be treated as medical waste when disposed.
13. Dispose the product if damaged or contaminated.
14. After using, wash it with medical cleaning agent and brush, then wash away foreign substances like adherent body fluid and body tissues completely.
15. Set the product to be cleaned by ultrasonic cleaner to avoid working part deterioration
16. Use this product with great care to avoid puncturing fingers because of its possession of sharp-edged part. Allowable engine speed: less than 800 rpm.
17. This product has possibility that be corrosive if sunk for a long time into sodium hypochlorite, EDTA, etc.

#### Storages and duration of use

1. Avoid storing at high temperature, humidity and direct sunlight. Keep liquids away. Store it at room temperature.
2. Do not damage or make a pinhole to packaging materials.
3. This product is subject to be improved without previous notice. Enforce first-in first-out method for stock management.
4. Do not store the instrument under a germicidal lamp to avoid deterioration

#### Method of sterilization:

The product you bought is already sterile.

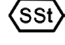
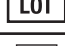
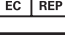
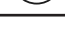
If you need to re-sterilize the product, put it in a sterilization pouch and place it on a sterilization tray for autoclave sterilization with reference to the following terms.

#### Terms for sterilization:

1. Sterilize by autoclave at 134°C while 18 minutes.
2. Do not use high pressure steam sterilizer which heats more than 200 degrees Celsius including drying process.
3. When reusing the instrument, wash away foreign substances completely and sterilize.

4. Regarding use of medical cleaning agent, follow the instruction manual by its manufacturer strictly.
5. Dispose the product if damaged or decreased in performance.

#### Symbols

	Handle Right angle RA	 800 - 1200 rpm	Recommended rotation Speed
	Stainless steel	 0 1 9 7	CE certificate
	Catalogue number		Manufacturer
	Batch code		Authorize representative in the European community
	Autoclavable at the specified temperature		Use-by date
	Consult instruction for use		Medical device
	Sterilized by radiation		Importer
	Do not use if packaging is damaged		

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