

MIRRORS **R&S**

INSTRUCTIONS FOR USE

CLEANING AND STERILIZATION INSTRUCTIONS

Use this instrument only for its intended purpose.

BEFORE THE INITIAL USE OF THE INSTRUMENT

Decontaminate before the first use and immediately after use.

CLEANING AND RINSING

Manual cleaning (recommended).

The use of a thermal disinfectant and ultrasonic cleaner is not recommended.

Rinse with demineralized clear water and dry (thorough inspection).

STERILIZATION

Packaging for sterilization. Autoclave sterilization (18 minutes at 134°C; 4 minutes at 132°C).

For delicate reflective surface dental mirrors,

avoid any contact with other instruments during these procedures.

Do not expose mirrors to temperatures exceeding 180°C.

Dental mirrors have a limited lifespan: no more than 100 sterilization cycles.

HANDLING

To prevent scratches on the mirror surface caused by other instruments,

clean and sterilize when fully disassembled,

and avoid contact with other instruments.

PLEASE NOTE THAT

Discoloration due to water spots is not a sign of corrosion.



stainless



PRODONT HOLLIGER
Groupe ACTEON
3 La Marnasse
63880 Olliergues
France
Tel : +33,04.73.95.56.42
Email : prodont-holliger@acteongroup.com



CADENCE
2 bis, chemin du Loup
93290 Tremblay-en-France - FRANCE