GUTTA PERCHA POINTS

- fr** Pointes de Gutta Percha
- en** Gutta Percha points
- it** Punti di guttapercha
- es** Puntas de guttapercha
- pt** Pontas de Guta Percha
- de** Guttapercha-Punkte
- nl** Gutta Percha Punten

fr

Utilisation

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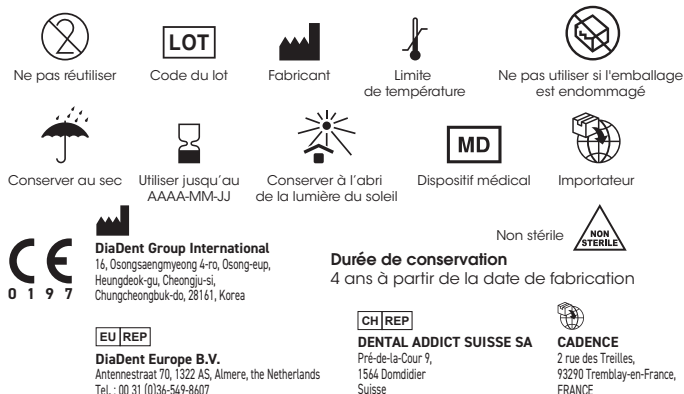
en

Purpose of use

Solid Gutta Percha used for root canal filling.

Preparation before use

1. Before using the product, check the expiration date and make sure there is no dirt, dust, or any foreign substance. Do not use if the Gutta Percha is too hard and not elastic.
2. Read and fully understand the manufacturer's instructions and study the step-by-step guide thoroughly.
3. Prepare all accessories and instruments required prior to the procedure.

Directions for use

In order to fill a root canal, a dental professional must follow the following steps.

1. Select a Gutta Percha master cone that fits the apex and check the working length before filling the root canal. Cut off the useless cone tip before inserting the Gutta Percha into the root canal.
2. When the Gutta Percha point is placed inside the root canal, take an x-ray to confirm that the Gutta Percha point is located 1mm away from the apex and perfectly fits the root canal.
3. Remove the Gutta Percha point from the root canal and dry with a paper point. Choose a file that is one size smaller than the one used to perforate the root canal. Apply the sealer to the file and spread the inside of the canal. Finish up by applying the sealer to Gutta Percha point tip around 1mm and inserting it into the root canal.
4. This procedure is the most fundamental method. However, a dental professional may consider using more Gutta Percha points or perform the procedure in a different way based on the patient's symptoms.

- Gutta Percha points are for a single use. Do not re-use them.

Maintenance after use

Gutta Percha points that are not used after the procedure must be stored in a container at room temperature to prevent contamination.

Handling & Storage

1. Store at room temperature.
2. Keep away from flammable and combustible materials and heat.
3. Manipulate with precaution during transportation.
4. Store separate from other products of different sizes and types.

Precautions for use

- General cautions and warnings
 1. The product must be used only by dental professionals. The directions for use must be followed carefully.
 2. Do not use on patients who are inflamed by or allergic to the latex.
 3. This product is for root canal use.
 4. Cover the patient's respiratory tract with a rubber dam before the procedure.
 5. Close the lid tightly after use.
- Cautions when using
 1. Check any secondary infection inside the root canal before placing the Gutta Percha Point.
 2. Use the same size and thickness of file and Gutta Percha point.
 3. Take an X-Ray after the procedure to ensure complete filling of the canal.

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DiaDent Europe B.V.