STERILE ABSORBENT PAPER POINTS

- (fr) Pointes de papier absorbant
- (en) Absorbent paper points
- (it) Punti di carta assorbente
- (es) Puntas de papel absorbente
- (pt) Pontas de papel absorvente
- (de) Absorbierende Papierspitzen
- (nl) Absorberende papieren punten

fr

Utilisation

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en

Purpose of use

An absorbent paper point is an endodontic paper point that is used during the endodontic therapy to dry, or remove any foreign materials from the root canal of a tooth.

Directions for use

There are two stages to remove the moisture, blood and other foreign substance from the root canal. A dental professional has to follow steps to fill the root canal.

1. Check the length of the root canal and adjust the paper point so that it fits to the apex of the root canal.
2. The selected paper point is inserted into the root canal to remove the blood or foreign substance before filling the root canal permanently with Gutta Percha Points.

The procedure is the most fundamental method. However, a dental professional may consider using more Absorbent Paper points to perform the procedure in a different way based on the patient's symptoms.

- Absorbent paper points are for single use. Do not re-use them.

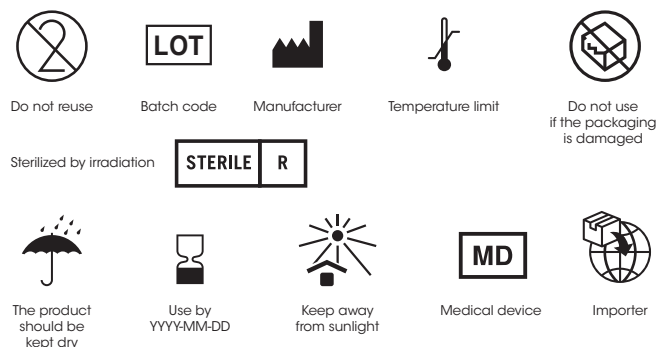
Handling & Storage

1. Store at room temperature (2-27 °C).
2. Do not throw the products while carrying.
3. Store separate from other products of different sizes and types.

Caution

1. Sterile Paper Points should be used with the same size and thickness of the Gutta percha points.
2. Avoid contact with moisture.
3. Disposable medical products.

Symbols

Shelf life
4 years from date of manufacture

CE
0 1 9 7
DiaDent Group International
16, Osongsangmyeong 4-ro, Osong-eup,
Heungdeok-gu, Cheongju-si,
Chungcheongbuk-do, 28161, Korea

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DiaDent Europe B.V.